# Hydrogel Pads

**Instructions for use**  
**Gebruiksaanweisung**  
**Mode d'emploi**  
**Istruzioni d'uso**  
**Gebruiksaanwijzing**  
**Instrucciones de uso**

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EN

## Instructions for use

Wash hands before using the product.  
 • Open pouch.  
 • Remove gel pad from the plastic liner.  
 • Gently place the gel side of the pad directly over your nipple.  
 • Do not use if the pouch is open or damaged.  
 • Prior to breastfeeding, remove the gel pad.  
 • Place on a clean surface.  
 • Gently wash the breast with warm water and pat dry.  
 • Then start breastfeeding.  
 • After breastfeeding, reapply the gel pad.  
*Do not use soaps or detergents to clean pad • Store gel pads in a cool, dry place • For more cooling sensation, gel pads can be stored in the fridge • Reusable for 24 hours • If gel pad becomes cloudy or milky, please discard and use a new one.*  
**Optional:** To improve soothing, briefly pass the gel side of the pad under a cool stream of water (1–2 seconds), shake off excess water and set aside for 2 minutes. Do not rinse more than twice a day.

## Important Safeguards

Use only as indicated • Keep out of reach of children, especially the plastic liner • Extra care should be taken if using with cracked nipples or fungal infection • If nipples are cracked, bleeding or have a burning sensation, contact your lactation consultant or physician • To prevent nipple pain it is important that baby feeds at mother's breast in a correct position.  
**Ingredients:** Water, glycerol, polymer

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© Medela AG 200.1726/2017-02/H